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**❖ REPORT 1: SHORT-TERM FOLLOW-UP / INFORME 1: SEGUIMIENTO A CORTO PLAZO**

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This study describes the short-term results regarding the implantation of 1 Avantgard model implant (GMI trademark) in a total of 1 patients (n=1), 1 men aged 31 years. Avantgard dental implant was certified in early 2016, as a consequence we are working to collect new dentists who want to collaborate with us in order to feed the corresponding PMS database, reporting clinical cases to study the success rate of implants Avantgard model manufactured by the Ilerimplant s.l. company.

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**DATA REPORTED / DATOS REPORTADOS**

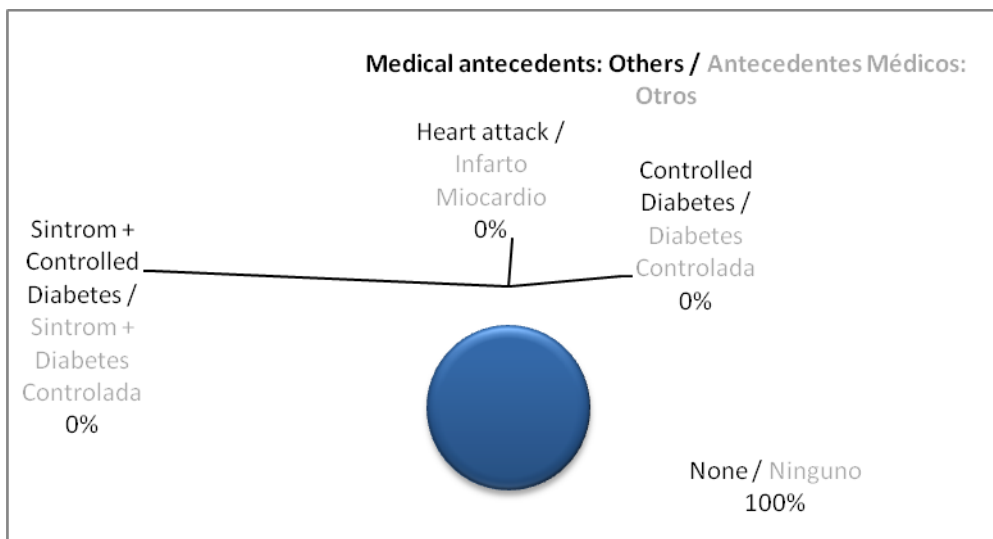
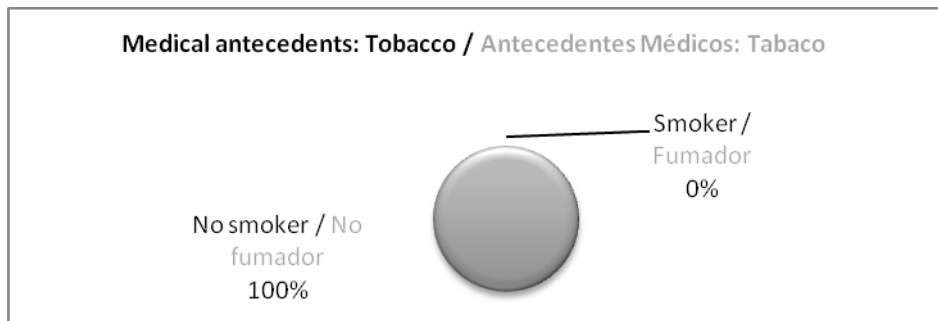
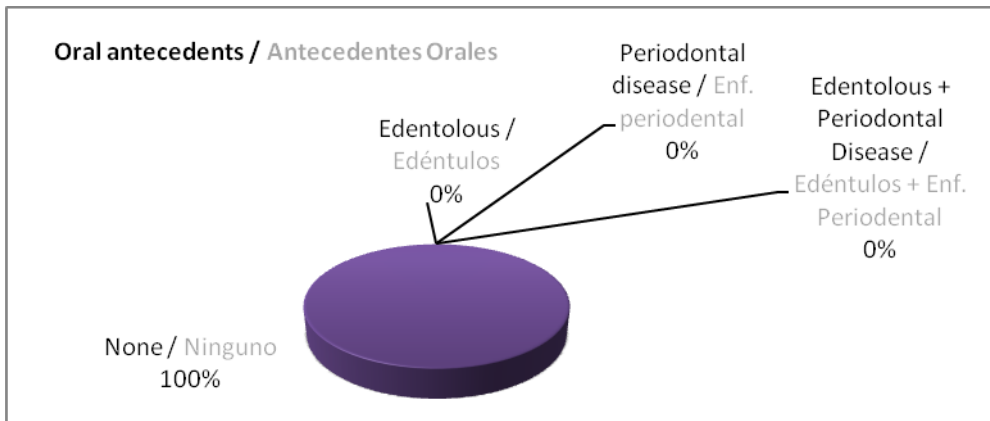
So far, there is a unique implantologist who used the follow-up record FO-SGC-011-16 provided by *Ilerimplant* company to carry out the data collection of the intervention carried out in July 2016. The collected data is exposed below:

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**Oral and medical history / Antecedentes orales y médicos:**

The patient did not have any oral history (e.g. edentulous, periodontal disease or both cases) and he was no smoker. Moreover, he did not have any other medical history such as (controlled diabetes, heart attack or sintrom).

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
**Range of implants / Gama de implantes**

The study was performed about the internal conical connection dental implants manufactured by the company Ilerimplant s.l., the following references are included in the study:

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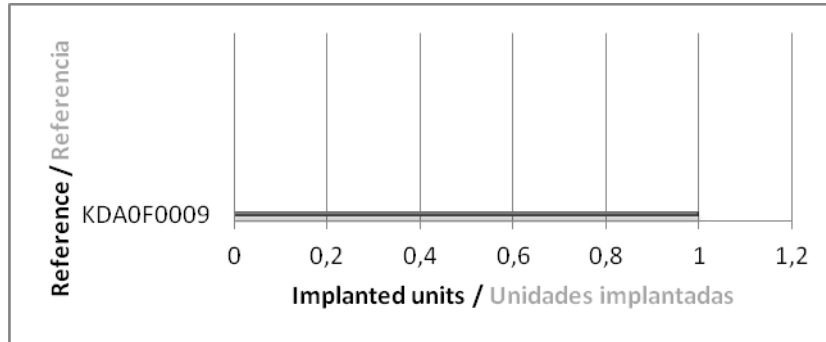The type of surgery used for the implant placements was:

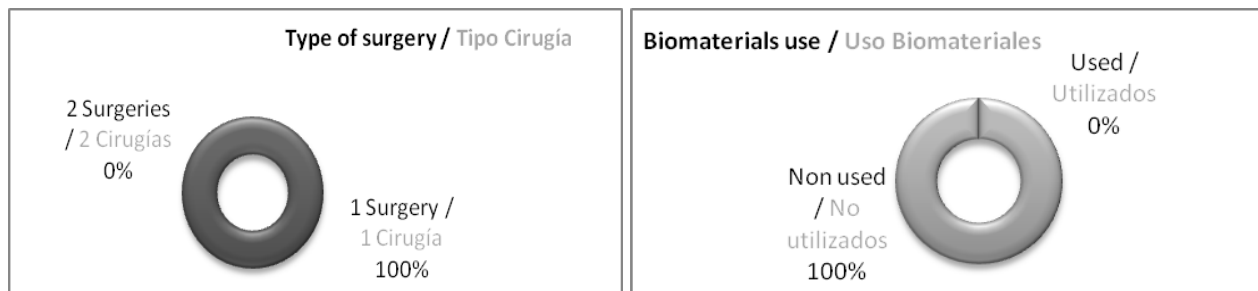
The implant was placed through a single surgery, so the surgical technique recommended by the company was not used.

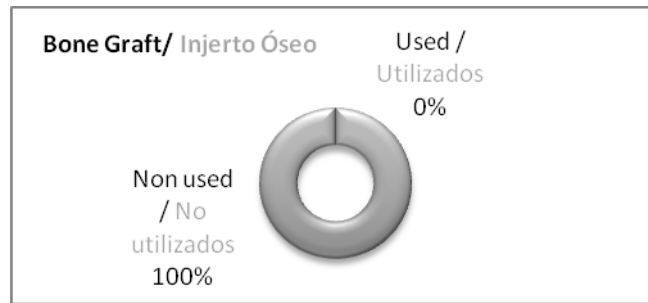
Neither biomaterials nor bone graft were used to help the implant integration.

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

The rehabilitation was carried out through a single crown.

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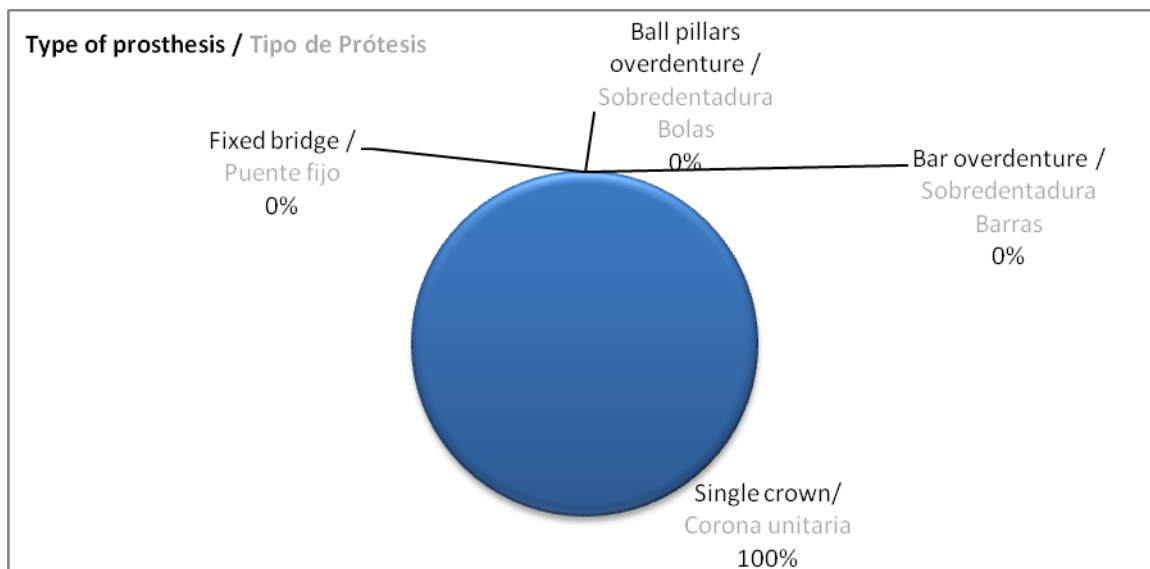Functional loading for the implant was classical.

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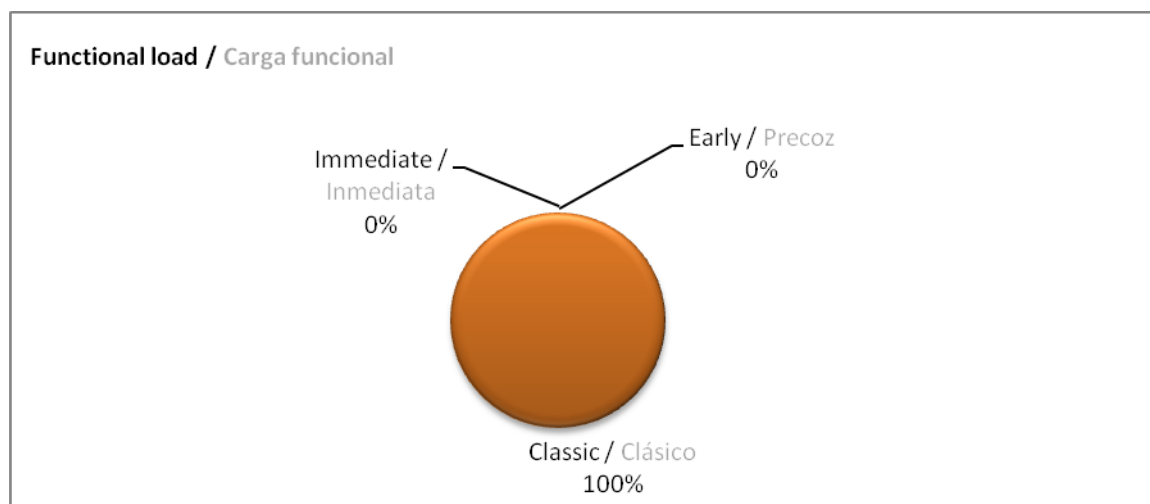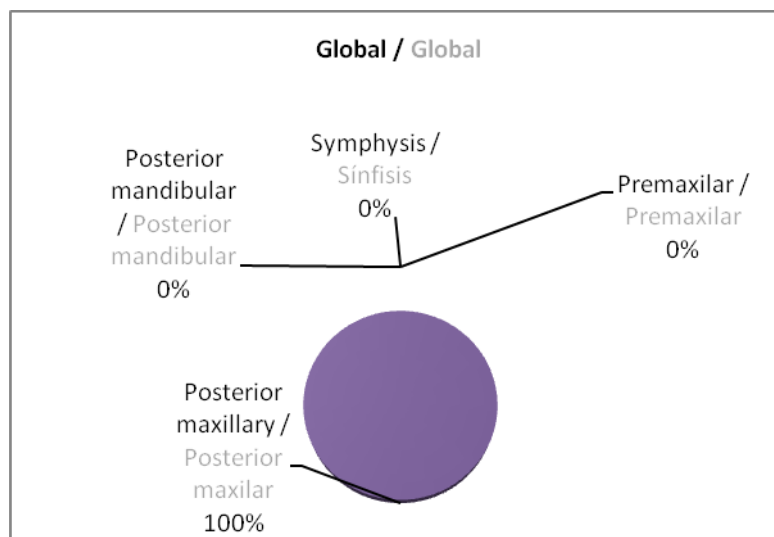
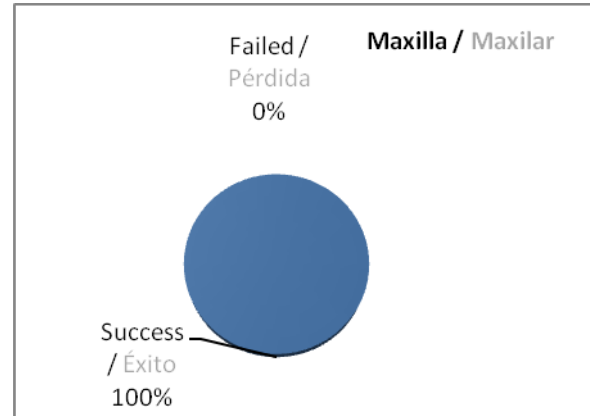
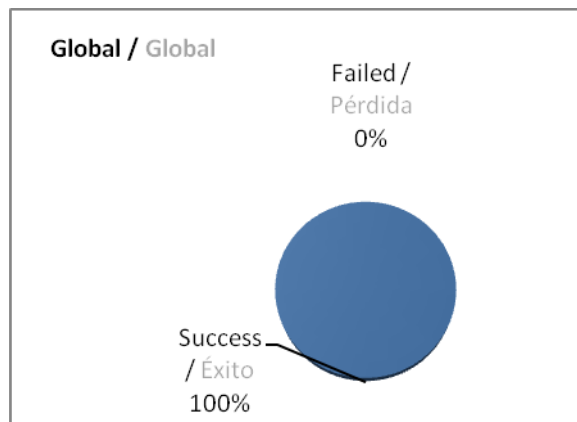
**GLOBAL SUCCESS RATE / TASA DE ÉXITO GLOBAL: 100%**

**CONCLUSIONS / CONCLUSIONES**

Currently, based on the data analyzed, Avantgard model implants of Ilerimplant S.L. company (GMI trademark) have a surgical rate similar to the rest of the successful implants available with similar features in the market. Remember that these results are only short-term success rates (placement-loading).

Based on the results obtained it does not considered to open any corrective and/or preventive action. We hope in the following reports have more cases to study, but by the time we do not draw firm conclusions because of the study is still not a representative sample.

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❖ **REPORT 1: LONG-TERM FOLLOW-UP / INFORME 1: SEGUIMIENTO A LARGO PLAZO**

It is needed more time to collect long-term data.

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**Table 1** – Correlation between follow-up time and number of implants.

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