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This study describes the short-term results regarding the implantation of 315 Frontier model implants (GMI trademark) in a total of 121 patients (n=2.60), 71 women and 47 men (information about 3 of them is not disposal), aged between 20 and 86 years. Over a period of 6 years 21 dentists reported clinical cases randomly to study the success rate of implants Frontier model manufactured by the Ilerimplant s.l. company.

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DATA REPORTED / DATOS REPORTADOS

The implantologists used the follow-up record FO-SGC-011-16 provided by *Ilerimplant* company to carry out the data collection of the 114 interventions carried out during the period June 2010 - June 2016. The collected data is exposed below:

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


Oral and medical history / Antecedentes orales y médicos:

25.62% (31 p.) of the patients treated were edentulous and 15.70% (19 p.) had periodontal disease. In the latter group, one patient was treated for diabetes, while another used Sintrom. Inside of both groups, 3 patients had both antecedents, they had been edentulous due to a periodontal disease. 39% of the implants studied are affected by the no favourable conditions.

On the other hand, 21% (26 p.) were smokers and according to these 26, eleven belonged to the edentulous group, six had periodontal disease and three to both groups.

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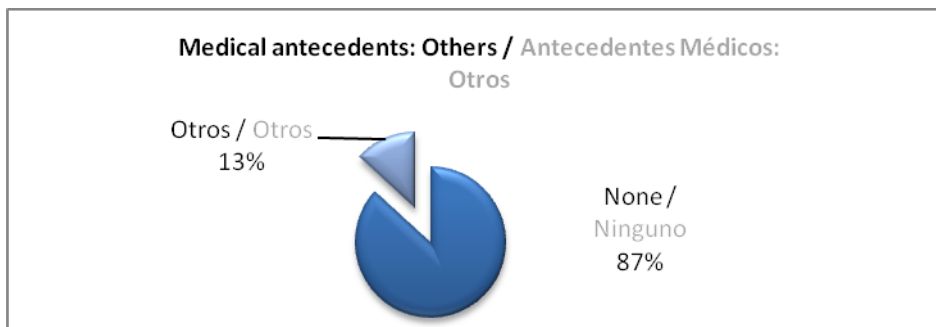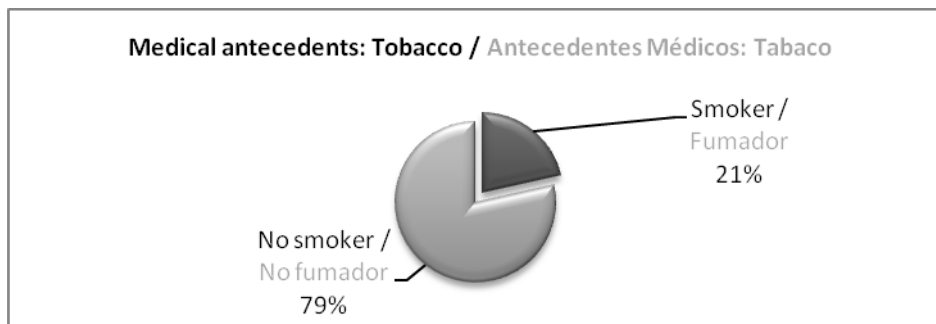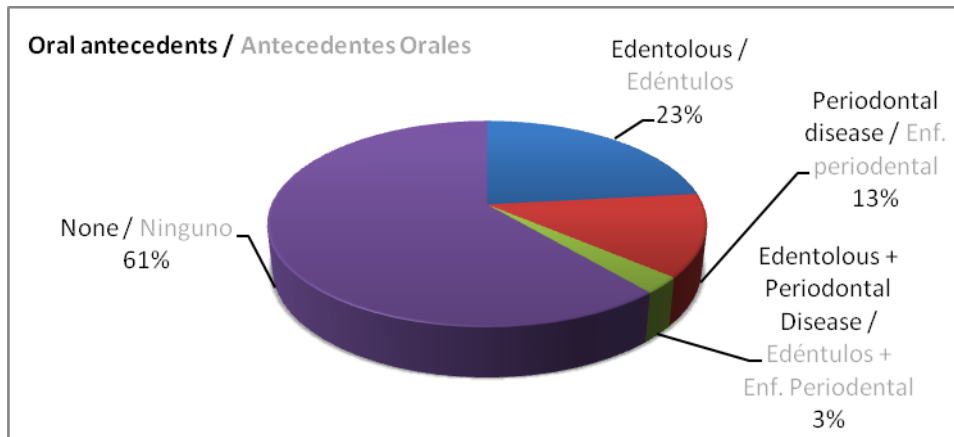
Range of implants / Gama de implantes

The study was performed about the internal hexagon dental implants manufactured by the company Ilerimplant s.l., the following references are included in the study:

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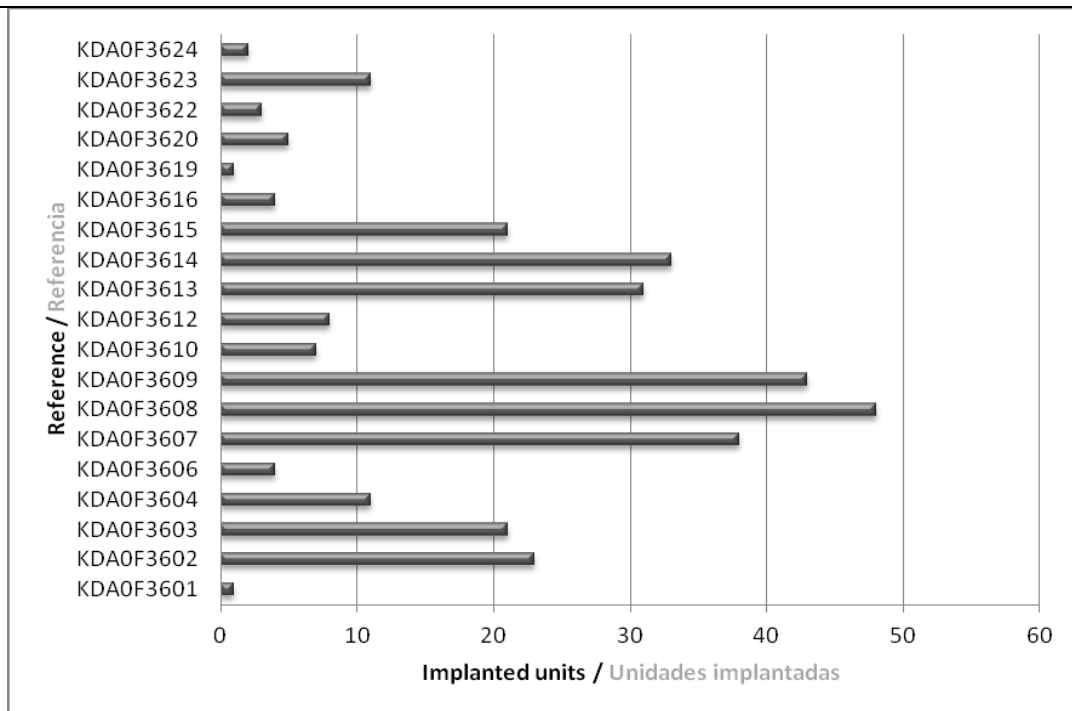References shaded in light blue are the measures most frequently use. Practically, the entire range of implants of the company are covered with data reported as you can see, except the Ø5.75 diameter implants due any follow-up have not received since its use is relegated to very specific cases. Even so, their inclusion for the correct evaluation of the implants response are not considered essential, since its characteristics are equivalent to the implants of different diameters, and therefore it can be expected a similar response.

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The type of surgery used for the 315 implant placements was:

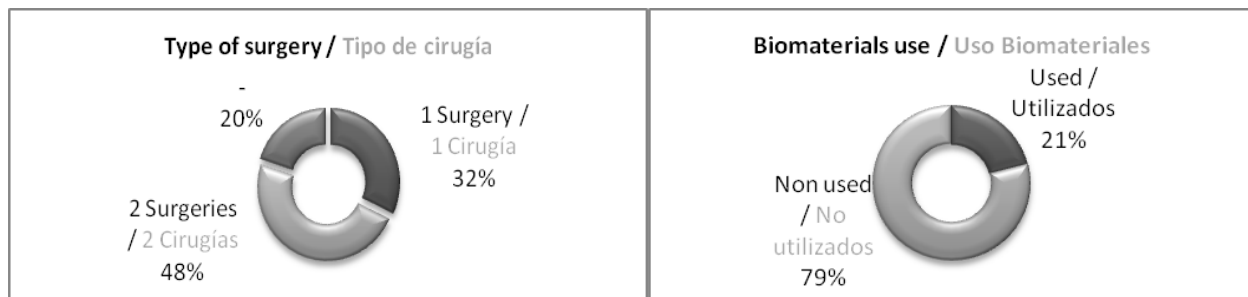
101 implants were placed through a single surgery and 151 were placed with 2 surgeries as our recommended surgical technique. We have no information on the remaining 20% (63 implants), but 14 implants were placed after extraction and none of them were charged immediately.

Biomaterials and bone graft were used to help the implant integration in 21% and 0% of the interventions respectively.

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

Regarding loaded implants, the rehabilitation of 115 implants was carried out through a single crown, 157 were rehabilitated through a fixed bridge, 19 with ball pillars overdenture and 21 with bar overdenture.

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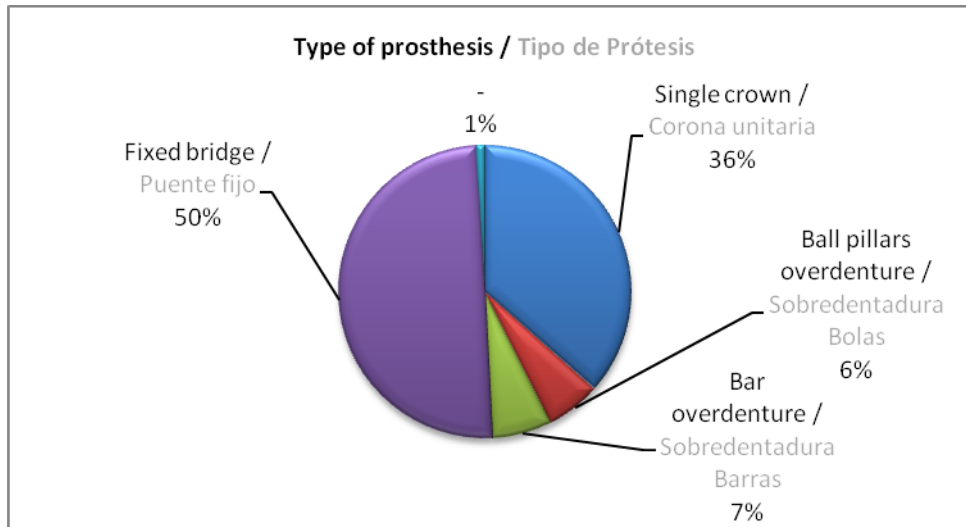Functional loading for 288 and 14 implants was classical and immediate respectively, except 1 early loading. While for the remaining 12, there are 6 implants which were not indicated the type of functional loading received (1 of them was explanted by no-osseointegration). The remaining 6 have not yet been loaded (4 of them suffered a loss and will be loaded once the new placed implants have been osseointegrated).

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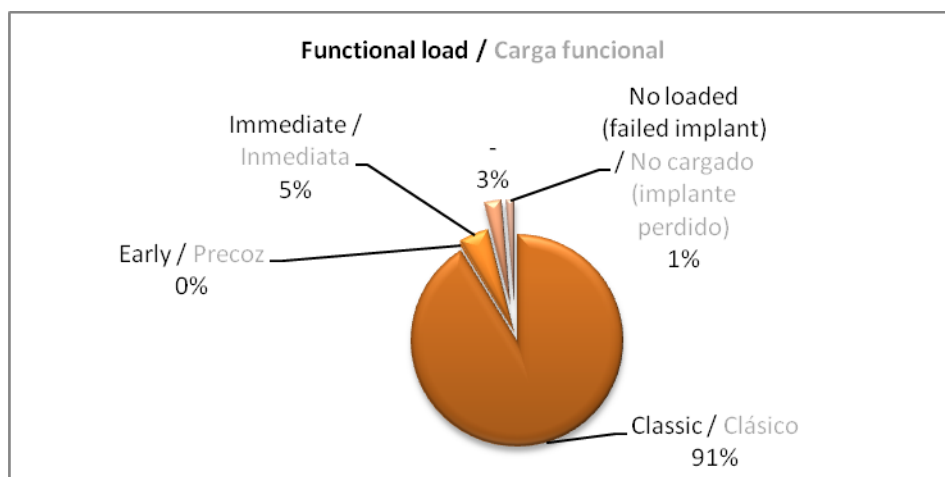
Regarding implants without information we do not know the exact details of placement about 7 of them which their information is not disposal. About remaining 18, we know that 8 were placed into the maxilla and the remaining 10 in the jaw.

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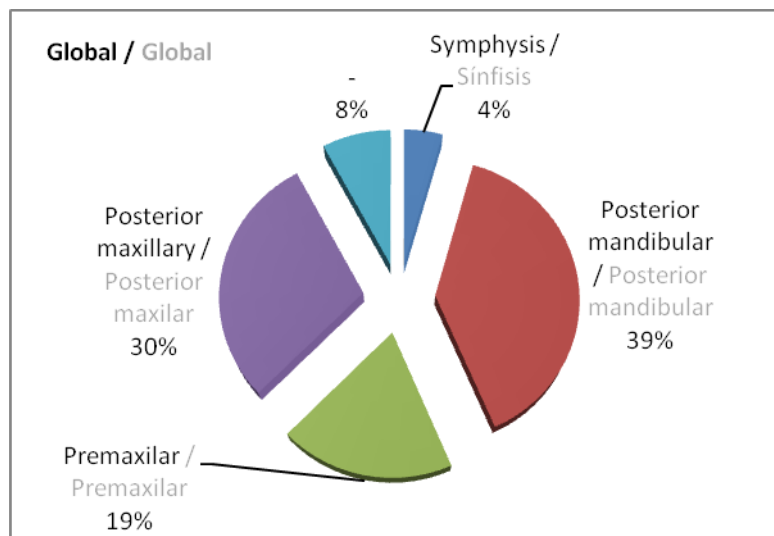
4 failed implant (2%)

Premaxilla: 3 failed implant.

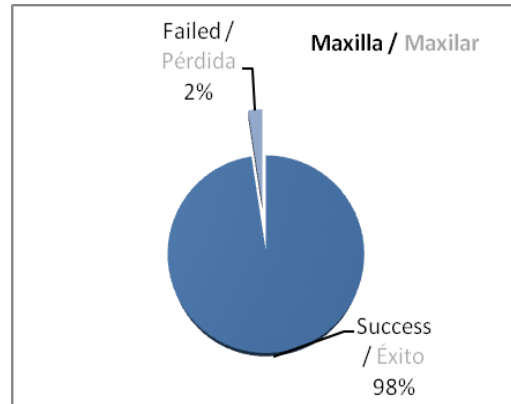1. Patient description: 67 years old and edentulous, 4 implants were placed, three of them were rehabilitated through a fixed bridge, one of them was failed (Implant placed at position 23 (Canine tooth) (KDAF3608 – Ø3.75 / L11.5) (2 surgeries).
2. Patient description: 57 years old with periodontal disease, 2 implants were placed, one of them was failed on second surgery (Implant placed at position 23 (Canine tooth) (KDAF3604 – Ø3.30 / L13) (2 surgeries).
3. Patient description: 43 years old, smoker and edentulous, 2 implants were placed, one of them was failed (1 surgery). Remark: The patient was suffered for unknowable diabetes after the placement of the first implant.

Posterior maxilla: 1 failed implant.

4. Patient description: 39 years old, smoker, 8 implants were placed, one of them was failed for lack of osseointegration after 4 months (Implant placed at position 17 (2nd molar) (KDAF3608 – Ø3.75 / L11.5) (1 surgery). We must note that this implant was not placed properly based on occlusal diagram of the company and it would suffer a fatigue failure surely. We can not do anything against it.

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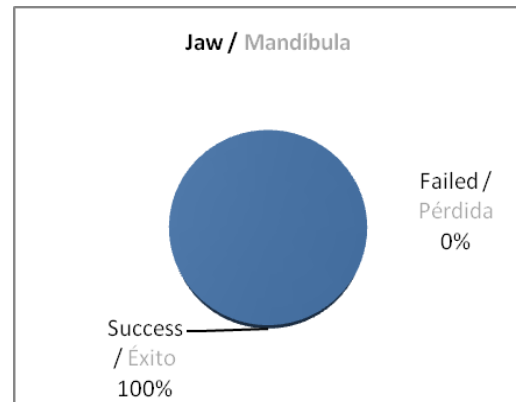
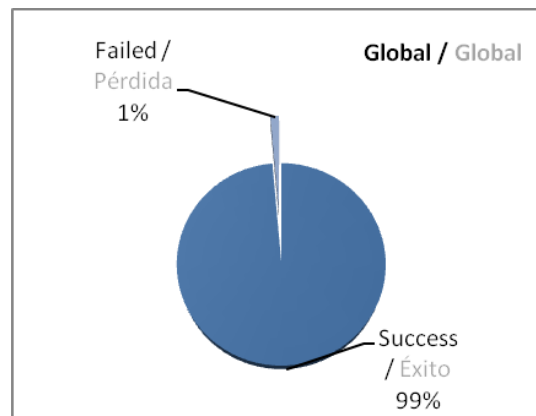
GLOBAL SUCCESS RATE / TASA DE ÉXITO GLOBAL: 99%

CONCLUSIONS / CONCLUSIONES

Currently, based on the data analyzed, Frontier model implants of Ilerimplant S.L. company (GMI trademark) have a surgical rate similar to the rest of the successful implants available with similar features in the market. Remember that these results are only short-term success rates (placement-loading).

Referring to the four implants lost we must indicate that were not implanted as per recommended occlusal diagram and validated by the company, which indicates that the diameter used for a canine tooth should be at least 4.25 mm, so the implants 1 and 3, it could consider as a possible cause of implant failure, an implant overload which prevent a proper osseointegration.

Frequency of use of different measures of implants according to this study:

- The diameter most widely used: 3.75 mm (44%)
- The length most widely used: 10 and 11.5 mm (both with 34 and 36% respectively%)

Based on the results obtained it does not considered to open any corrective and/or preventive action. We hope in the following reports have more cases to study, but by the time, according to 315 cases analyzed it is showed that our implant works properly and that its success rate is excellent and considering that patients with unfavourable antecedents for an implant placement were analysed in 39% cases.

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❖ **REPORT 3: LONG-TERM FOLLOW-UP / INFORME 3: SEGUIMIENTO A LARGO PLAZO**

After evaluation of the proper osseointegration and implant charging in the short-term follow-up, where 4 of 315 implants failed during the osseointegration stage, inducing 99% of success rate, a long-term product follow-up has been developed, and the outcomes obtained by the time are attached below there are:

Regarding the 311 implants charged properly, it has lost the follow-up of 66 (25 of 121 patients), due to patients have not attended established revisions by dentist, this involves 20.66% loss of sample follow-up.

The rest of 245 implants work properly as per information provided by doctors. Obtaining a 100% of success rate in the second stage of the study for those implants has performed the follow-up.

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*Patient with both succeeded and failed implants. / Pacientes con implantes exitosos y fracasados.

Table 1 – Correlation between follow-up time and number of implants.

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Must be also mentioned that the results obtained in the study performed by University of Sevilla were attached in the long-term follow-up report nº 1 issued on 01 / 09 / 2014.

- **Early loading of GMI implants with internal connection and sandblasted-acid-etched surface.** Velasco Ortega E., Monsalve Guill I., Matos Garrido N., Jiménez Gerra A., García Méndez A., Medel Soteras A., Ortiz García I., España López A.

Published in the *AVANCES en Periodoncia e Implantología* (Volume 26 – Number 2 – August 2014).

Study results: After 1 clinical follow-up year and implant post-loading, the success rate of the GMI Frontier implants was 99.1%.

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