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❖ **REPORT 5: SHORT-TERM FOLLOW-UP / INFORME 5: SEGUIMIENTO A CORTO PLAZO**

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This study describes the short-term results regarding the implantation of 98 Phoenix model implants (GMI trademark) in a total of 35 patients (n=2.8), 22 women and 13 men, aged between 32 and 82 years. Over a period of 6 years 14 dentists reported clinical cases randomly to study the success rate of implants Phoenix model manufactured by the Ilerimplant s.l. company.

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DATA REPORTED / DATOS REPORTADOS

The implantologists used the follow-up record FO-SGC-011-16 provided by *Ilerimplant* company to carry out the data collection of the 35 interventions carried out during the period June 2010 - June 2016. The collected data is exposed below:

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Oral and medical history / Antecedentes orales y médicos:

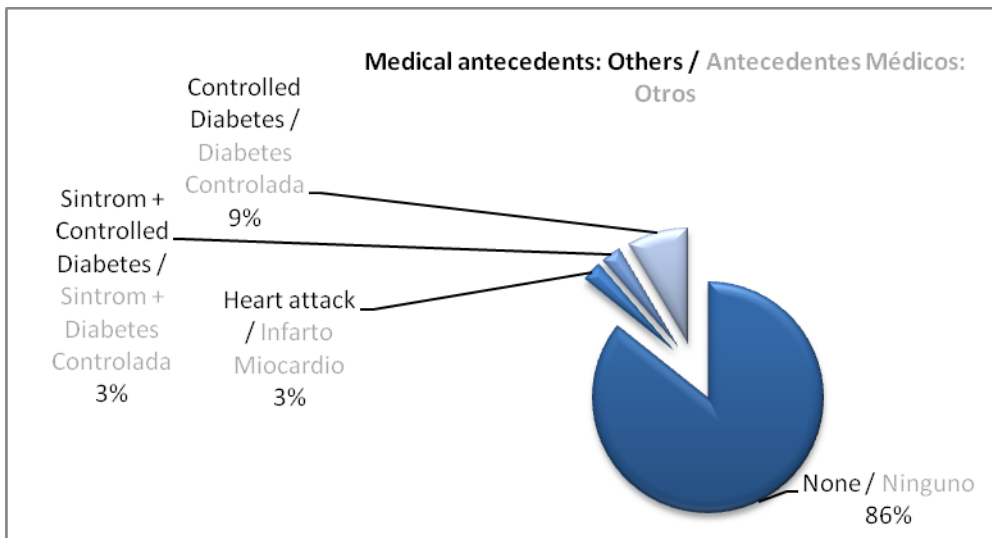
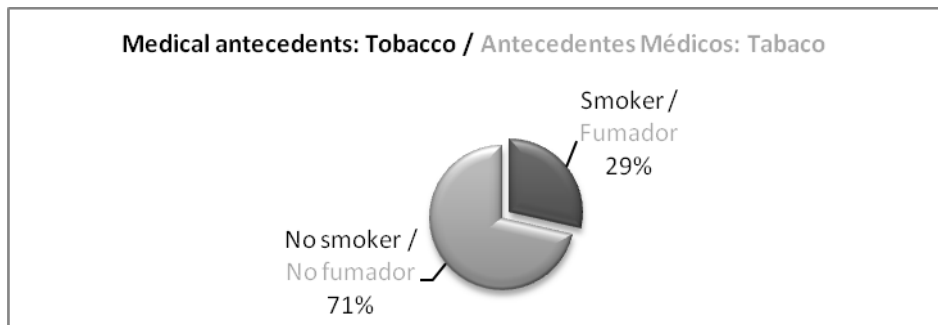
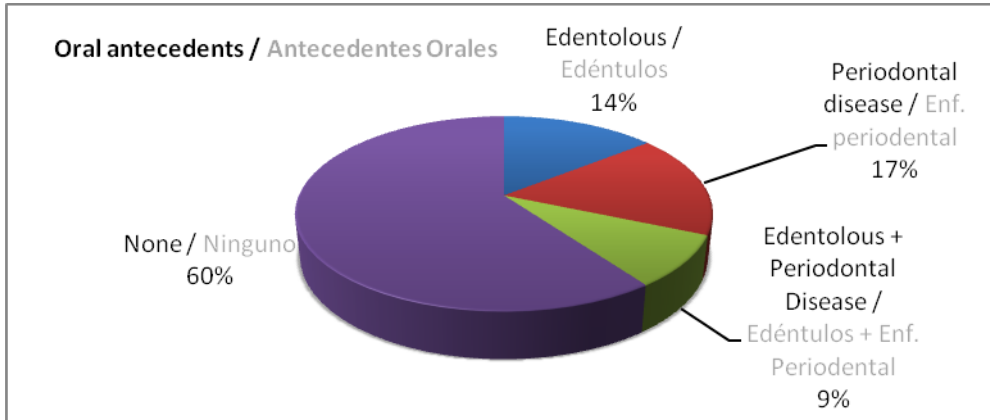
22.86% (8 p.) of the patients treated were edentulous and four of them were diagnosed and treated for diabetes, furthermore one of this four used Sintrom. Another 25.71% (9 p.) had periodontal disease. Inside of both groups, 3 patients had both antecedents, they had been edentulous due to a periodontal disease. In the latter group, one patient had also suffered a heart attack and other diabetes. 40% of the implants studied are affected by the no favourable conditions.

On the other hand, 28.57% (10 p.) were smokers and according to these 10, three belonged to the edentulous group, one had periodontal disease, two to both groups and the rest of all did not suffer any antecedent.

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
antecedente.

Range of implants / Gama de implantes

The study was performed about the external hexagon dental implants manufactured by the company Ilerimplant s.l., the following references are included in the study:

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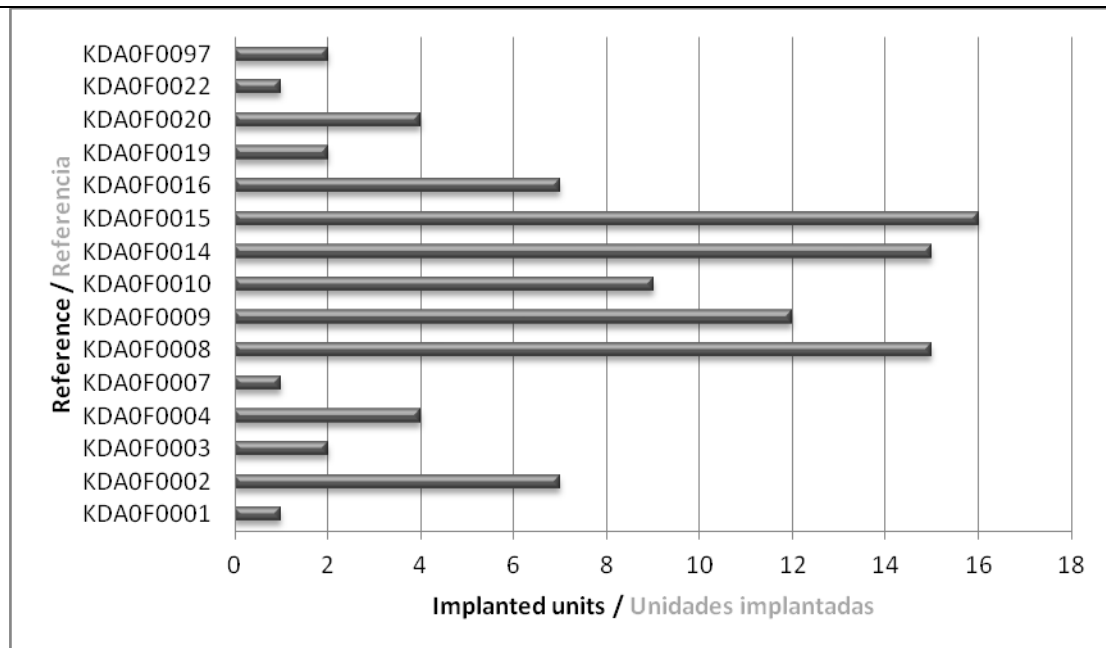References shaded in light blue are the measures most frequently use. Practically, the entire range of implants of the company are covered with data reported as you can see, except the Ø6.00 diameter implants due any follow-up have not received since its use is relegated to very specific cases. Even so, their inclusion for the correct evaluation of the implants response are not considered essential, since its characteristics are equivalent to the implants of different diameters, and therefore it can be expected a similar response.

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The type of surgery used for the 98 implant placements was:

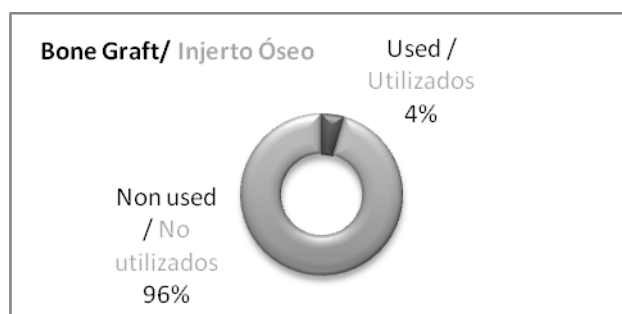
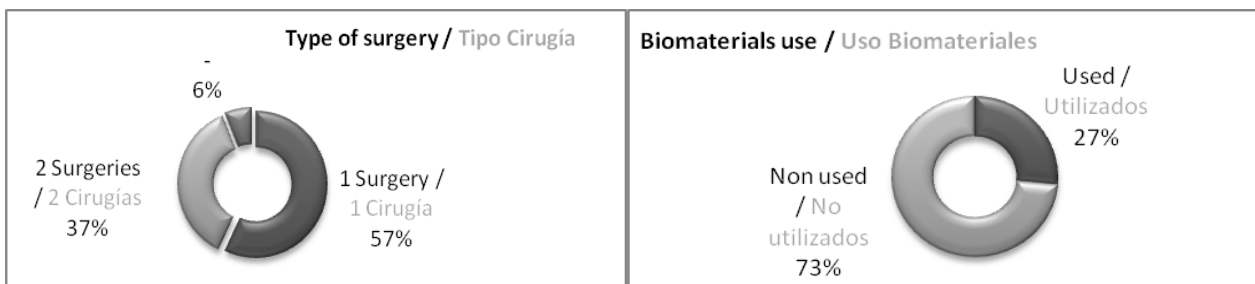
56 implants were placed through a single surgery and 36 were placed with 2 surgeries, so 57% of the interventions did not use the surgical technique recommended by the company. We have no information on remaining 6% (6 implants), but 6 of them were placed after extraction and none of them were charged immediately.

Biomaterials and bone graft were used to help the implant integration in 27% and 4% of the interventions respectively.

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

One implant was never osseointegrated and therefore was not loaded. Regarding loaded implants, the rehabilitation of 22 implants was carried out through a single crown, 49 were rehabilitated through a fixed bridge, 15 with ball pillars overdenture and 10 with bar overdenture.

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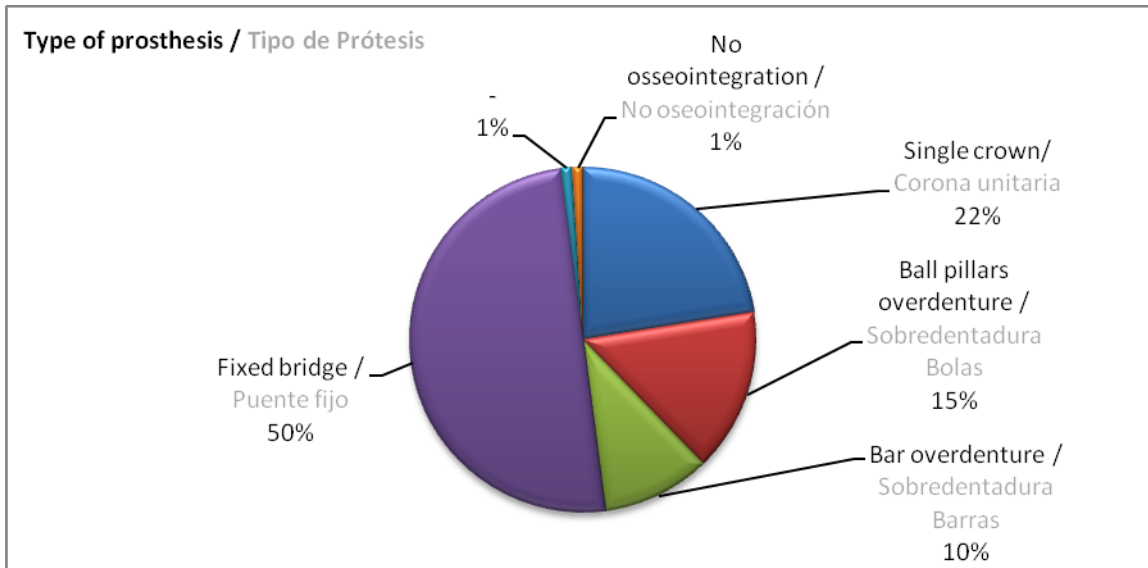Functional loading for 74 and 22 implants was classical and immediate respectively, while for the remaining 2, one has no information and the other was not loaded (failed implant).

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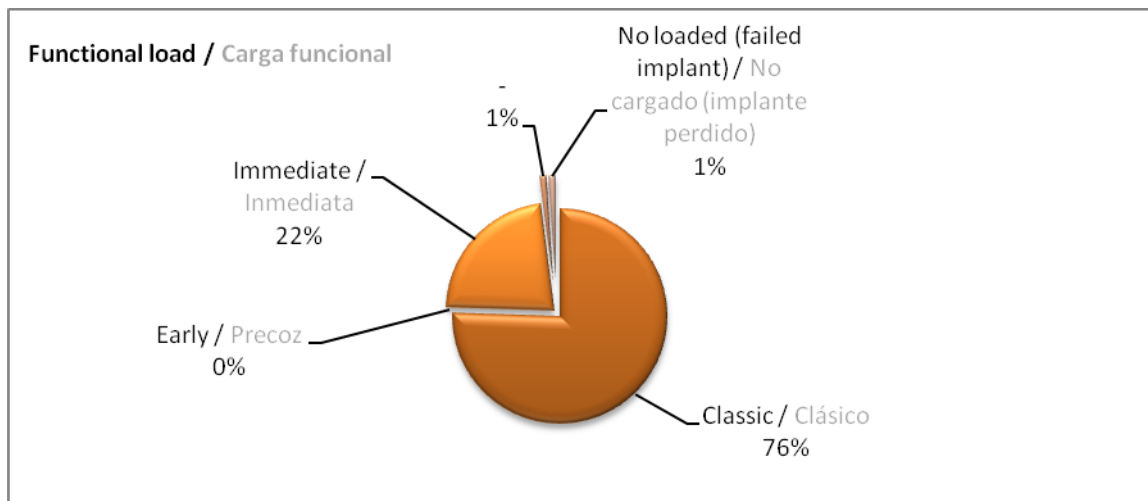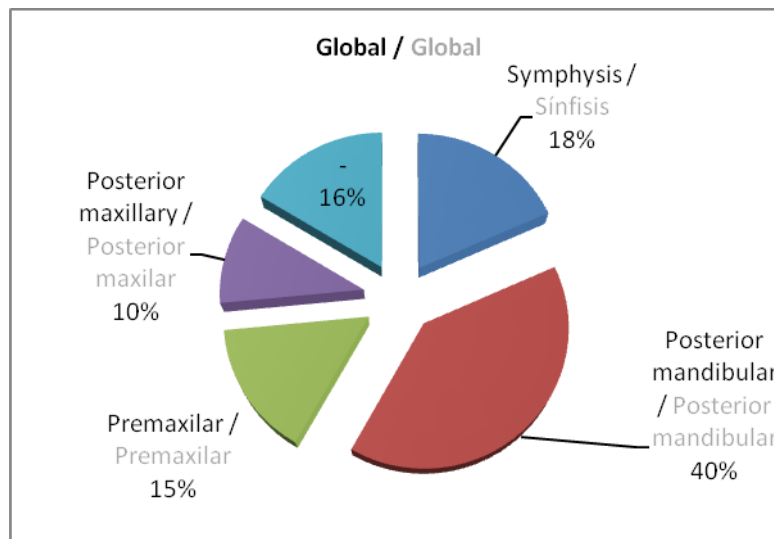
We do not know the exact details of placement about 16 implants which their information is not disposal, but we know that 7 were placed into the maxilla and the remaining 9 in the jaw.

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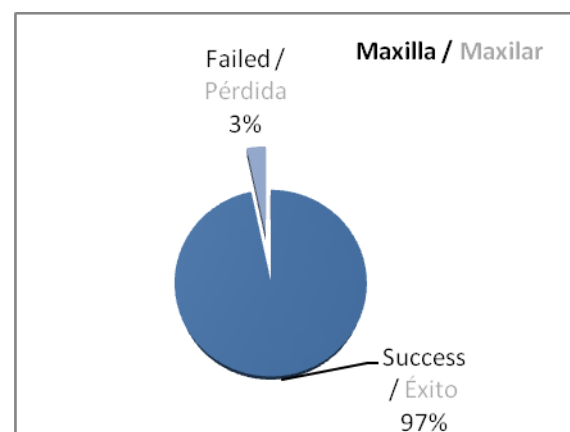
1 failed implant (3%)

Premaxilla: 1 failed implant

1. Patient description: 62 years old, smoker and with periodontal disease, 10 implants were placed, one of them was lost for lack of osseointegration. (Implant placed at position 13 (Canine tooth) (KDAF0014 - Ø4.0 / L10) (1 surgery).

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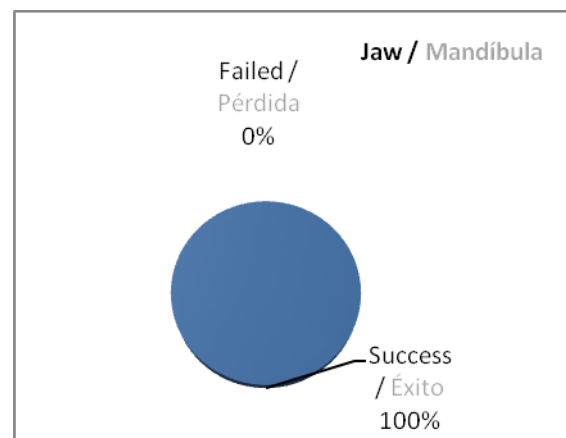
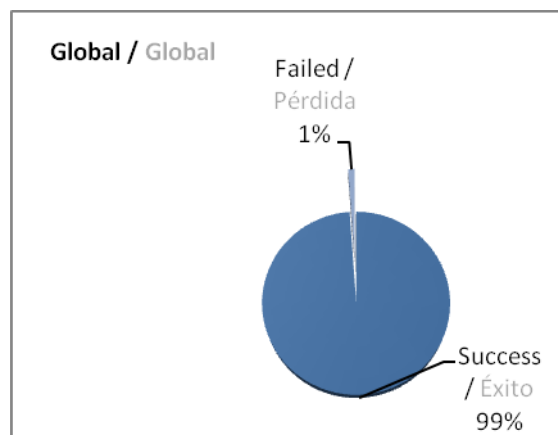
GLOBAL SUCCESS RATE / TASA DE ÉXITO GLOBAL: 99%

CONCLUSIONS / CONCLUSIONES

Currently, based on the data analyzed, Phoenix model implants of Ilerimplant S.L. company (GMI trademark) have a surgical rate similar to the rest of the successful implants available with similar features in the market. Remember that these results are only short-term success rates (placement-loading).

Concerning the implant lost and based on the patient's history, we should note that only 1 of 10 implants placed was failed, this is considered a big success.

Frequency of use of different measures of implants according to this study:

- The diameter most widely used: 4.0 mm (39%)
- The length most widely used: 10 mm (42%)

Based on the results obtained it does not considered to open any corrective and/or preventive action. We hope in the following reports have more cases to study, but by the time, according to 98 cases analyzed it is showed that our implant works properly and that its success rate is excellent and considering that patients with unfavourable antecedents for an implant placement were analysed in 40% cases.

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❖ **REPORT 3: LONG-TERM FOLLOW-UP / INFORME 3: SEGUIMIENTO A LARGO PLAZO**

After evaluation of the proper osseointegration and implant charging in the short-term follow-up, where 1 of 98 implants failed during the osseointegration stage, inducing 99% of success rate, a long-term product follow-up has been developed, and the outcomes obtained by the time are attached below there are:

Regarding the 90 implants charged properly, it has lost the follow-up of 14 (4 of 35 patients), due to patients have not attended established revisions by dentist, this involves 15,55% loss of sample follow-up.

The rest of 76 implants work properly as per information provided by doctors. Obtaining a 100% of success rate in the second stage of the study for those implants has performed the follow-up.

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**Patient with both succeeded and failed implants. / Pacientes con implantes exitosos y fracasados.*

Table 1 – Correlation between follow-up time and number of implants.

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